NUTRICIA
flocare[®] Infinity[™] III

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NUTRICIA

NUTRICIA
flocare® Infinity™ III

SETTING UP THE FLOCARE® INFINITY™ III FEEDING PUMP

AT FIRST PUMP START-UP AND AFTER A FACTORY RESET (E.G. FOLLOWING REPAIR OR SERVICING).

- 1 Home button 2 ON/OFF button
 3 Soft button 1 4 Soft button 2 5 Soft button 3

The **Flocare® Infinity™ III feeding pump** must be set up before first pump start-up after a factory reset. This means setting the language, time and date. The steps that need to be followed to do this are explained below.

1 Turn the pump on by pressing and holding the *ON/OFF button* on the right-hand side for 2 seconds.

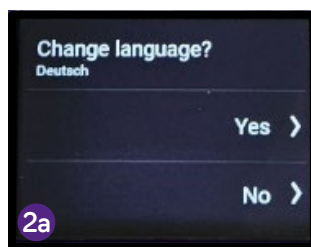
2 The language selection will then appear on the display. At first pump start-up or after a factory reset, English will appear first in the list.

You can scroll through the language menu and select a language using *soft buttons 1 and 3* on the right-hand side, next to the arrows.

2a The following **18 languages** are available for selection: English, Czech, Danish, German, Greek, Spanish, French, Italian, Hungarian, Dutch, Norwegian, Polish, Portuguese, Russian, Finnish, Swedish, Turkish, Arabic.

Press the button on the right-hand side, next to the word “YES”, to confirm the selection.

3 After this, it is necessary to **set the time**. To do this, press the button on the right, next to the word “Hours”.

3a The current hour can be set by pressing *soft buttons 1 and 3* on the right, next to the “+” and “-” symbols.

To confirm, press the button on the right-hand side, next to the **hours** set (taking this example, next to the number “14”).

3b To set the minutes, press the button on the right, next to the word “Minutes”.

The current minute can be set by pressing *soft buttons 1 and 3* on the right, next to the “+” and “-” symbols. To confirm, press the button on the right, next to the minutes set.

4 **Setting the date** comes next.

To do this, press the button on the right, next to the word “Year”.

4a The current year can be set by pressing *soft buttons 1 and 3* on the right, next to the “+” and “-” symbols. To confirm, press the button on the right, next to the **year number** set.

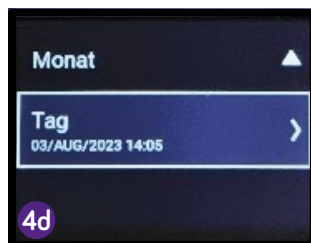
4b Next, you will be asked to set the current **month**. As when setting the year number, enter the current month using *soft buttons 1 and 3* on the right, next to the “+” and “-” symbols.

4c To confirm, press the button on the right, next to the month set (taking this example, next to the number “08”).

4d Finally, you must set the current day. As when setting the month, enter the current day using *soft buttons 1 and 3* on the right, next to the “+” and “-” symbols. To confirm, press the button on the right, next to the day set.

5 Once the setup is complete, you must again press the *HOME button* at the top left-hand side.

5a This will take you to the home screen, storing the settings that have been entered. You can then switch off the pump by pressing the *ON/OFF button*.

You can find more information on your Flocare® pump and on the full range of feeding tubes, giving sets, pumps and other accessories, as well as training videos, safety information and instructions for use, at <https://nutricia.cc/ernaehrungspumpen>

Subject to change; errors excepted. No liability will be accepted for any printing errors.

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